

# Dandle PAL™



## INTENDED USE / CLINICAL BENEFIT:

The Dandle PAL™ is a flexible positioning aid that can be used in a variety of ways. The PAL has two internal bags filled with silky micro disks: a larger bag that fills the horseshoe and a smaller bag near the twill strap.

## WARNINGS / CONTRAINDICATIONS:

- Hospital use only.
- Special caution should be taken to avoid placing the full weight or too many micro disks on top of small infants.
- Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.
- To prevent cross-contamination between patients, clean according to hospital policies.
- If micro disks should ever become loose or if a hole occurs in the fabric cover, discard immediately.

## CLEANING INSTRUCTIONS:

The Dandle PAL is washable and reusable product.

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



## INSTRUCTIONS FOR USE:

The Dandle PAL is a flexible positioning aid that can be used in a variety of ways.

- The horseshoe part, or legs, can be placed around the baby's head or feet to create a boundary, help to maintain a position, or give a feeling of security.
- The smaller bag can be placed on top of the larger one if a higher boundary is needed. The smaller bag can be held in place by pulling the strap under the horseshoe part.
- The horseshoe can be opened up wide and put behind a baby in side-lying position to help maintain the position and keep the baby in flexion. The smaller bag may be strapped on top of the larger one to help keep the baby from rolling back.
- The smaller bag can be used to prop a pacifier or support tubing.
- The micro disks can be moved in the legs so the legs are very full (fold the unused portion underneath) or not so full depending on the baby's medical and developmental needs.

## STORAGE AND DISPOSAL:

No special requirements.

*Not made with natural latex rubber. DEHP free.*

Please contact Dandle·LION Medical with any questions or comments  
or to report any issue with the use of our product.

Rev 7:0721

