

Dandle PAL™ - Wipeable



DEHP-free, BPA-free, not made with any natural rubber latex.

The Dandle PAL is a wipeable, reusable positioning aid that can be used for hospitalized preterm and ill infants in prone, supine and side-lying positions.

WARNINGS / CAUTIONS:

- **Hospital use only.**
- **Special caution should be taken to avoid placing the full weight or too many micro discs on top of small infants.**
- **Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.**
- **To prevent cross-contamination between patients, clean according to hospital policies.**
- **To prevent growth of any internal contaminant, regularly inspect the wipeable covering for any visible signs of damage or wear. Discard if necessary.**
- **Always use with cover.**

CLEANING INSTRUCTIONS:

- The Dandle PAL is a wipeable and reusable product that should be wiped clean with a standard hospital quaternary disinfectant any time it becomes soiled and between patient use.
- The cloth covers can be machine washed and dried at temperatures up to 71°C. Non-chlorine bleach is recommended.
- The non-woven covers are single patient use and should be discarded appropriately.

INSTRUCTIONS FOR USE:

The Dandle PAL is a flexible positioning aid that can be used in a variety of ways. The PAL has two internal 'beanbags': a large bag that fills the horseshoe and a smaller bag at the rounded top.

- The horseshoe part, or legs, can be placed around the baby's head or feet to create a boundary, help to maintain a position, or give a feeling of security.
- The smaller beanbag can be placed on top of the larger one if a higher boundary is needed. The smaller beanbag can be held in place by pulling the strap under the horseshoe part. The horseshoe can be opened up wide and put behind a baby in side-lying position to help maintain the position and keep the baby in flexion.
- The smaller beanbag can be used to prop a pacifier or support tubing.
- The micro discs can be moved in the legs so the legs are very full (fold the unused portion underneath) or not fully depending on the baby's medical and developmental needs.

STORAGE AND DISPOSAL:

No special requirements.

WARRANTY:

The Dandle PAL has a useful life of 18 months from date of manufacturer (found on tag).

Please contact Dandle•LION Medical with any questions or comments or to report any issue with the use of our product.

Rev2:0721