

Mini PAL™ - Wipeable

Not made with natural latex rubber. DEHP free.




INTENDED USE / CLINICAL BENEFIT:

The Mini PAL is a flexible positioning aid that can be used in a variety of ways. The Mini PAL has an outer bag and an internal bag that is filled with silky micro discs.

WARNINGS / CAUTIONS:

- **Hospital use only.**
- **Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.**
- **To prevent cross-contamination between patients, clean according to hospital policies.**
- **To prevent growth of any internal contaminant, regularly inspect the wipeable covering for any visible signs of damage or wear. Discard if necessary.**
- **Always use with cover.**
- **If the outside of the product becomes punctured or otherwise compromised, discard immediately.**

CLEANING INSTRUCTIONS:

- The Mini PAL is a wipeable and reusable product that should be wiped clean with a standard hospital quaternary disinfectant any time it becomes soiled and between patient use.
- The disposable cover is single patient use and should be discarded immediately. 

For Mini PAL cloth covers (Available Accessory):

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



INSTRUCTIONS FOR USE:

The Mini PAL is a flexible positioning aid that can be used in a variety of ways for all babies and was developed intentionally for babies < 800 g. When used inside the XS Dandle® ROO2, Dandle® ROO Lite, or Dandle® WRAP Stretch, the Mini PAL will fill the space at the bottom of the pouch area to improve product functionality.

- In supine, with the baby's head up against the headroll, place the larger base piece ("body") in the space from the baby's bottom to where the pouch meets the base. The baby's legs should rest on top of the "body" in a posterior pelvic tilt. Use the Mini PAL "legs" to snuggle up beside the torso, under the arms, filling up space and helping to keep hips and shoulders in flexion. With the Mini PAL inside the positioner, the wings and pouch will align properly.
- With the baby in a side-lying position, place the "body" behind the baby and put the lightweight "legs" over the baby's shoulders and hips to maintain position or provide a sense of comfort and security.
- May be used with larger babies to help support proper positioning, provide proprioceptive feedback, or support tubing/equipment.
- If no headroll is available and midline head positioning is warranted, place Mini PAL around infant's head with "legs" beneath baby's shoulders, supporting arms in flexion.
- If limb extension is necessary, place the "body" under the joint and the "legs" above and below the joint to hold it in extension.

STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments or to report any issue with the use of our product.

Rev 1:0323

