Dandle® PAL

Adaptable Boundary with Innovative Dual Weighted Pouches

The Dandle PAL:

JAL 1

Comes in two styles to meet each unit's needs:

- Wipeable with disposable or reusable cloth covers
- Washable

Serves a variety of functions for babies and caregivers:

- Acts like an extra set of hands for comforting baby during patient care
- Supports proper placement of medical devices to reduce the risk of malposition, unintended removal, or pressure injury
- Works well alone, in multiples, or in combination with other positioning aids to meet a variety of clinical needs

Caution: Do not place the entire Dandle PAL on top of a baby.

The Dandle PAL is a flexible positioning accessory that can be used alone, in pairs or as a complement to any other positioning aid.

Two separate pouches in the Dandle PAL are filled with tiny plastic micro discs that feel silky to the touch. This unique two pouch design allows the height and stability of the Dandle PAL to be increased or decreased to best meet each baby's needs.

When the pouches are stacked on one another, the twill strap stabilizes them in place, keeping support secure. The long 'legs' of the Dandle PAL can provide support to flexion around the baby's trunk, while giving ample room for the micro discs to be shifted around as needed to increase or decrease height or volume. This flexible design makes the Dandle PAL a versatile positioning aid.



Dandle

Dandle[®] PAL





Two Dandle PALs are used to provide containment and boundaries entirely around the baby. Notice the 'legs' are long enough to help round the shoulders and keep the hips in flexion.



The Dandle PAL keeps the head in midline while providing support to the trunk and rounding the shoulders. Pouches are stacked one on top of the other and secured with the strap, ensuring a firm, stable surface at the desired height.



Two PALs encourage a flexed posture while supporting optimal position of the CPAP. Using multiple PALs in this way allows full visibility of the infant while providing soothing, neurodevelopmentally-supportive positioning.



The smaller pouch is folded on top of the large pouch to provide firmness and height, which helps keep baby snugly in place without sliding down the inclined mattress.

D 11505 B10	Dandle PAL- Washable	10/case	Reusable printed cotton covers for Dandle PAL- Wipeable	25/case
D 20505 B10	Dandle PAL- Wipeable, with white disposable cover	10/case	White disposable covers for DandlePAL- Wipeable	25/case