

is a reusable positioning boundary that can be used for hospitalized infants.

Para

WARNINGS:

- Do not place near infant's face or use in any manner that might restrict the airway.
- Always use with a cover.
- Keep 'loop' at end of cover away from infant's fingers or toes.
- Hospital use only.
- DO NOT stick needles into the Cozy Cub through the cover.

CAUTIONS:

- Special caution should be taken to avoid placing any part of the Cozy Cub on top of any part
- Regularly inspect the Cozy Cub for any visible signs of damage or wear such as cuts, tears, or stains.
- Immediately discard if Cozy Cub has any visible signs of stains, damage or wear.

WARRANTY: The Cozy Cub has a useful life of 18 months from date of manufacture (found on

S:

able with a standard hospital quaternary disinfectant and reusable. The printed cloth covers are washable and

E:

b is a flexible positioning aid that can be used for infants with a variety of medical conditions. Regardless of p

c. Midline

e. 360-degree proprioception

d. Ability for free movement and recoil

tainment while allowing for full visualization of the infant. It can be used as a complete boundary or folded away. The loop at the end of the cover can be used to make a more secure, smaller boundary by pulling the opposing end through the loop. This provides more or less support for infant or any medical equipment.

or partial boundary around the infant. Be sure to support shoulders, hips and legs to encourage proper flexion. The Cozy Cub is used to prevent infant from rolling over, to keep hands to face, and to provide flexion in the lower extremities. The head and neck are in alignment, trunk is supported, and hips are drawn into pelvic tilt. Use the Cozy Cub to maintain