



## Dandle PAL™

Infant Positioning Aid  
Made without natural rubber latex  
DEHP free

The Dandle PAL™ is a reusable positioning aid that can be used for hospitalized preterm and ill infants in prone, supine and side-lying positions.

Pictorial instructions are available on-line at [www.dandleLIONmedical.com](http://www.dandleLIONmedical.com).

### WARNINGS:

- **Hospital Use Only.**
- **Special caution should be taken to avoid placing the full weight or too many beans on top of small infants.**
- **Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.**
- **To prevent cross-contamination between patients, launder according to hospital policies.**
- **If beans should ever become loose or if a hole occurs in the fabric cover, discard immediately.**

### CLEANING INSTRUCTIONS:

The Dandle PAL is a washable and reusable product.



Machine wash and dry at temperatures up to 71°C.  
Non-chlorine bleach is recommended.

### INSTRUCTIONS FOR USE:

The Dandle PAL is a flexible positioning aid that can be used in a variety of ways.

- The horseshoe part, or legs, can be placed around the baby's head or feet to create a boundary, help to maintain a position or give a feeling of security.
- The smaller beanbag can be placed on top of the larger one if a higher boundary is needed. The smaller beanbag can be held in place by pulling the handle under the horseshoe part.
- The horseshoe can be opened up wide and put behind a baby in side-lying position to help maintain the position and keep the baby in flexion.
- The smaller beanbag can be used to prop a pacifier or support tubing.
- The beans can be moved in the legs so the legs are very full (fold the unused portion underneath) or not so full depending on the baby's medical and developmental needs.

rev4:0916