

Dandle•LION Gel Pillow

Latex and DEHP free




Dandle•LION™
Medical

The Dandle•LION Gel Pillow may be used in conjunction with the Dandle ROO Systems or any time a supportive pillow or surface is needed for a hospitalized infant. The Gel Pillow bags are made from a soft, pliable plastic and are filled with a water-based, non-toxic food grade gel that is resistant to bacterial and fungal growth. Each gel pillow comes with a cotton cover that can be washed and re-used.

WARNINGS:

- For hospital use only
- Do not lie baby face down on gel in a way that could restrict baby's airway. Use with medical supervision.
- If baby requires thermal support, prewarm gel in incubator or warmer. Do not microwave.
- Always use with cloth cover.
- Do NOT X-RAY through gel. Remove before x-ray.
- Single patient use (ⓧ)

CLEANING INSTRUCTIONS:

- Gel Pillow: To prevent cross contamination between patients, gel pillows are indicated for single patient use. Any reuse of the pillows is not warranted by the manufacturer and should be made only after an appropriate method of sanitizing/disinfecting has been validated by the user.
- Gel Pillow Cover: Machine wash and dry at temperatures up to 160°F.   

INSTRUCTIONS FOR USE:

- The Dandle•LION Gel Pillows provide a soft, pliable surface to support and cradle a baby's head and/or body. For a fuller pillow, push the gel to one side, fold part of the bag underneath and tape.
- After extended use in a warm environment, the gel will begin to breakdown and the bag will become puffy. Please discard the bag when this occurs.

1214: Rev.5

