

Dandle•LION Gel Pillow

DEHP-free, BPA-free, not made with any natural rubber latex




Dandle•LION™
Medical

The Dandle•LION Gel Pillow may be used in conjunction with the Dandle ROO or ROO2 or any time a supportive pillow or surface is needed for a hospitalized infant or child. The Gel Pillow bags are made from a soft, pliable polyethylene film and are filled with a water-based, non-toxic food grade gel that is resistant to bacterial and fungal growth. Each gel pillow comes with a cotton cover that can be washed and re-used or a disposable cover that should be discarded after use.

WARNINGS:

- For hospital use only
- Do not lie baby face down on gel in a way that could restrict baby's airway. Use with medical supervision.
- If baby requires thermal support, prewarm the gel in an incubator or warmer. Do not microwave.
- Always use with cloth cover.
- Do NOT X-RAY through gel. Remove before x-ray.
- Single patient use (ⓧ)

CLEANING INSTRUCTIONS:

- Gel Pillow: To prevent cross contamination between patients, gel pillows are indicated for single patient use. Any reuse of the pillows is not warranted by the manufacturer and should be made only after an appropriate method of sanitizing/disinfecting has been validated by the user.
- Gel Pillow Cotton Cover: Machine wash and dry at temperatures up to 160°F.   

INSTRUCTIONS FOR USE:

- The Dandle•LION Gel Pillows provide a soft, pliable surface to support and cradle a baby's head and/or body. For a fuller pillow, push the gel to one side, fold part of the bag underneath to achieve desired fullness.
- After extended use in a warm environment, the gel will begin to break down. The gel will become watery and the bag will become puffy. Please discard the pillow when this occurs.

0415: Rev.6

