

Dandle•LION Gel Pillow

DEHP-free, BPA-free, not made with any natural rubber latex






The Dandle•LION™ Gel Pillows provide a soft, pliable surface to support and cradle a baby's head and/or body. The Gel Pillows are appropriately sized for use in the Dandle ROO2 and may help to reduce the need for frequent re-positioning. The pillows are made from a soft polyethylene film filled with a non-toxic, water based food gel that is resistant to bacterial and fungal growth. Available in 8 sizes and each Gel Pillow comes with a cotton cover that can be washed and re-used between patients. Disposable covers are also available.

WARNINGS:

- For hospital use only
- Do not lie baby face down on gel in a way that could restrict baby's airway. Use with medical supervision.
- If baby requires thermal support, prewarm the gel in an incubator or warmer. Do not microwave as hot spots may form.
- Always use with cover.
- Do NOT X-RAY through gel as air pockets may distort x-ray results.
- Single patient use (⊗)

CLEANING INSTRUCTIONS:

- Gel Pillow: To prevent cross contamination between patients, gel pillows are indicated for single patient use. Any reuse of the pillows is not warranted by the manufacturer and should be made only after an appropriate method of sanitizing/disinfecting has been validated by the user.
- Gel Pillow Cotton Cover: Machine wash and dry at temperatures up to 160°F (71°C).   

INSTRUCTIONS FOR USE:

- Prewarm the Gel Pillow in the incubator or under hot running water if necessary. Gel will maintain the ambient temperature of the environment. Place the pillow where needed. For a fuller pillow, push the gel to one side, fold part of the bag underneath to achieve desired fullness.
- After extended use in a warm environment, the gel will begin to break down. The gel will become watery and the bag will become puffy. Please discard the pillow when this occurs.

STORAGE

- Store below 75°F (24°C).

1120: Rev.8

