

Polly PAL™

Not made with natural latex rubber. DEHP free.



INTENDED USE / CLINICAL BENEFIT:

The Polly PAL is a flexible positioning aid that can be used in a variety of ways. It is filled with plastic micro discs, that feel silky to the touch.

WARNINGS / CAUTIONS:

- **Hospital use only.**
- **Special caution should be taken to avoid placing the full weight or too many micro discs on top of small infants.**
- **Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.**
- **To prevent cross-contamination between patients, clean according to hospital policies.**
- **If micro discs should ever become loose or if a hole occurs in the fabric cover, discard immediately.**

CLEANING INSTRUCTIONS:

The Polly PAL is a washable and reusable product.

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



INSTRUCTIONS FOR USE:

The Polly PAL is a flexible positioning aid that can be used in a variety of ways.

- Place around the baby's head or feet to create a boundary, help to maintain a position, or give a feeling of security.
- Use it to prop a pacifier or support tubing.
- Move the micro discs in different places to make parts or sections fuller or less full to accommodate equipment or the baby's positioning needs.

STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments or to report any issue with the use of our product.

Rev 1:1122



DandleLION Medical
www.dandleLIONmedical.com

22 Shelter Rock Lane
203-791-9001

Danbury, CT 06810
© DandleLION Medical 2021