


Snuggly ROO™

Not made with natural rubber latex. DEHP and BPA free.

The Snuggly ROO is a neurodevelopmentally supportive positioning aid that can be used for hospitalized preterm and ill infants in prone, supine, or side-lying positions.

**WARNINGS:**

- **Hospital use only.**
- **Do not place straps around the baby's neck or face or use in any manner that might restrict infant's airway.**

	<p><b>CLEANING INSTRUCTIONS:</b></p> <p>Machine wash and dry all cloth components at temperatures up to 71° C. Higher temperatures will cause additional shrinkage. Visually inspect product after each laundering.</p>
---	---

**INSTRUCTIONS FOR USE:**

- Select the correct size Snuggly ROO according to the infant's weight and body proportions. See label for size recommendations by weight.
- Put the Snuggly ROO in the incubator, warmer or crib with the footroll at the bottom of the bed and the straps off to the sides.
- Place the baby in the Snuggly ROO in a prone, supine or side-lying position so the baby's legs are flexed appropriately and tucked inside the footroll, and the head and spine are in neutral alignment.
- Place the baby's hands near the face in a flexed position.
- Pull one strap over the baby's shoulder and arm to help hold the baby's hands in a comfortable, flexed position near his face.
- Tuck the strap under the Snuggly ROO so the weight of the baby will hold it in place.
- Pull the strap on the opposing side over the other shoulder and arm to assist with maintaining appropriate flexion and tucker under. In side-lying, the second strap may not be needed.
- In order to maintain the desired position, the caregiver may use a Polly PAL, Dandle PAL or Cozy Cub to give the baby support at the head and keep the baby from kicking out of the footroll.

- Check all lines and tubes after positioning the baby to ensure proper and secure placement.

**NOTE:** If greater visualization of the baby is needed, the straps may be placed off to the side.

**DISPOSAL AND STORAGE:**

No special requirements.

Please contact DandleLION medical with any questions or comments or to report any issue with the use of our product.

DandleLION Medical 22 Shelter Rock Lane Danbury, CT 06810

[info@dandlelionmedical.com](mailto:info@dandlelionmedical.com) | [www.dandleLIONmedical.com](http://www.dandleLIONmedical.com) | 203-791-9001

© DandleLION Medical 2011

Rev1:0621