

# Dandle PAL™ - Wipeable

Not made with natural latex rubber. DEHP free.



The Dandle PAL is a wipeable, reusable positioning aid that can be used for hospitalized preterm and ill infants in prone, supine and side-lying positions.

## WARNINGS / CAUTIONS

- Hospital use only.
- **Special caution should be taken to avoid placing the full weight or too many beans on top of small infants.**
- **Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.**
- **To prevent cross-contamination between patients, clean according to hospital policies.**
- **To prevent growth of any internal contaminant, regularly inspect the wipeable covering for any visible signs of damage or wear. Discard if necessary.**
- **Always use with cover.**

## CLEANING INSTRUCTIONS

- The Dandle PAL is a wipeable and reusable product that should be wiped clean with a standard hospital quaternary disinfectant any time it becomes soiled and between patient use.
- The cloth covers can be machine washed and dried at temperatures up to 71°C. Non-chlorine bleach is recommended.
- The non-woven covers are single patient use and should be discarded appropriately.

## INSTRUCTIONS FOR USE

The Dandle PAL is a flexible positioning aid that can be used in a variety of ways. The PAL has two internal 'beanbags': a large bag that fills the horseshoe and a smaller bag at the rounded top.

- The horseshoe part, or legs, can be placed around the baby's head or feet to create a boundary, help to maintain a position, or give a feeling of security.
- The smaller beanbag can be placed on top of the larger one if a higher boundary is needed. The smaller beanbag can be held in place by pulling the strap under the horseshoe part. The horseshoe can be opened up wide and put behind a baby in side-lying position to help maintain the position and keep the baby in flexion.
- The smaller beanbag can be used to prop a pacifier or support tubing.
- The beans can be moved in the legs so the legs are very full (fold the unused portion underneath) or not fully depending on the baby's medical and developmental needs.

## WARRANTY

- The Dandle PAL has a useful life of 18 months from date of manufacturer (*found on tag*).

Rev1:620