



Tranquilo® Soothing Mat

Not made with natural latex rubber. DEHP free.



The battery operated Tranquilo Soothing Mat is an effective tool for soothing fussy and hard to console babies. The four vibration levels and two additional pulsatile rhythm choices are easily changed by the push of a button. The Tranquilo Soothing Mat is distributed to hospitals by DandleLION Medical.




WARNINGS / CAUTIONS

- **Hospital use only.**
- **Do not stick needles into the Tranquilo Soothing Mat as this may compromise the integrity of the covering and create a contaminaton/infection risk.**
- **Before each use, inspect this product for damage. Discontinue use if any parts are missing or broken.**
- **Battery caution:**
 - **Do not mix old and new batteries.**
 - **Do not mix alkaline, standard (carbon-zinc), or rechargeable (nickel-cadmium, lithium-ion) batteries.**
 - **Dispose of batteries properly.**
 - **Securely close battery compartment and zip cover to prevent unwarranted access to the batteries.**

CLEANING INSTRUCTIONS

- Mat may be wiped down with standard hospital disinfectant between patients. Do not use phenolic based disinfectants.
- White disposable covers are single patient use and should not be reused between patients.

INSTRUCTIONS FOR USE

- Four AA batteries are required for use. Unzip the battery pocket located at the end of the mat. Remove the battery box and place 4 AA batteries inside the compartment. Follow the “+/-“ signs for proper installation. Place the battery box back into the pocket and close the zipper.
- Put on disposable cover.
- Hold the Power Button (identified by the  symbol) for 3 seconds to turn on the Mat for continuous use
- OR -
- Continue to hold down the Power Button for five seconds after vibrations begin to enable a 60-minute automatic shut-off function.
- Select mode by pressing the Mode button (identified by “M” symbol) until the desired mode is activated.
- MODES in order:
 - Extra low vibration
 - Low vibration
 - Medium vibration
 - High vibration
 - Low vibration with slow pulsing beat
 - High vibration with fast pulsing beat
- Place infant on top of the Mat.
- Turn off the Mat by pressing and holding the Power Button for 3 seconds.

View our video instructions here.



WARRANTY

- One year limited warranty.
- Expected life of the motor is 2000 hours.

Rev1:820