

Cleaning Instructions:

- The Spry Flo-Form positioner should be disinfected by wiping the casing with a hospital-approved disinfectant. (Suggested max 72% alcohol, max 1000ppm active chlorine.) Iodine and blood stains can be removed with a hospital-approved stain remover.
- After the disinfection, the positioner should be wiped down using a clean cloth moistened with clean water. Allow the pillow to dry fully before use.
- Bleaching, scrubbing, machine washing, tumble drying, dry cleaning, autoclaving, or ironing are not recommended.

If you have a specific protocol that our suggested cleaning doesn't fall under, please let us know, and we can determine if it will negatively affect the life of the product. The product should be cleaned as needed during use.