



# Flo-Form<sup>®</sup> Positioners

Fluidized Technology for Custom Care

## Flo-Form Positioners:

- Allow for multipatient use (after cleaning + disinfecting)
- Made with a soft fluidized medium that is easily shaped
- Come in a variety of sizes and formats
- Covers available in washable cloth or disposable non-woven fabric
- DEHP and latex free

Dandle•LION's Flo-Form positioners, utilizing Spry Therapeutics' fluidized technology, support and protect the position of the developing infant, especially when handling is not well-tolerated.

The fluidized medium allows the caregiver to create, adjust and maintain the infant's desired position while providing an individualized, supportive environment. This proprietary technology facilitates indirect manipulation of the infant's body, without the need for physical handling.

The Flo-Form<sup>®</sup> positioners can be used in a variety of ways:

### For head positioning:

- Allows the caregiver to redistribute pressure, protect the skin and promote a healthy head shape
- Promotes neutral head / trunk alignment
- Simplifies tube / line management

### For body positioning:

- Stabilizes and maintains position during periods of acute or critical illness, including admission, surgery, ECMO or hemodynamic instability
- Supports a flexed, midline position while promoting neutral body alignment
- Redistributes pressure over a greater surface area, minimizing friction and shear
- Provides close-fitting containment that can be conformed to each baby's unique shape

**Dandle•LION<sup>®</sup>**  
MEDICAL

# Flo-Form® Positioners



A Dandle® WRAP Stretch provides developmentally-supportive positioning while a Flo-Form Positioner maintains the head in midline.



Flo-Form Positioners fit snugly against the infant's body, supporting midline flexion.



The Dandy Prone Pad™ facilitates proper prone positioning, while a Flo-Form Tube stabilizes the baby's position and provides proprioceptive input.



The close-fitting containment provided by the Flo-Form Positioner helps to stabilize and maintain the 'C' curve of the spine.

FF707	Flo-Form Positioner with Washable Cloth Cover, 7" x 7"	18/box
FF7010	Flo-Form Positioner with Washable Cloth Cover, 7" x 10"	12/box
FF9015	Flo-Form Positioner with Washable Cloth Cover, 9" x 15"	6/box
FF12020	Flo-Form Large Positioner with Washable Cloth Cover, 12" x 20"	8/box
FF16024	Flo-Form Large Positioner with Washable Cloth Cover, 16" x 24"	6/box
FF20028	Flo-Form Full Body Positioner with Washable Cloth Cover, 20" x 28"	4/box
FFT30	Flo-Form Positioning Tube with Washable Cover, 30" x 6.5"	6/box
FFT40	Flo-Form Positioning Tube with Washable Cover, 40" x 6.5"	6/box
FFT55	Flo-Form Positioning Tube with Washable Cover, 55" x 6.5"	6/box