

Flo-Form[®] Positioner

Not made with natural latex rubber. DEHP free.



INTENDED USE / CLINICAL BENEFIT:

The Flo-Form[®] Positioner uses a fluidized medium that maximizes contouring and support. The product is designed to aid in providing developmental positioning for neonatal patients while redistributing pressure on sensitive skin.

WARNINGS / CAUTIONS:

- Hospital use only.
- Do not use if positioner has been torn or punctured.
- Clean with hospital approved disinfectant.
- Place a barrier, such as a cover or sheet, between the positioner and the patient's skin.
- Do not use in any manner that might restrict the patient's airway.
- Use of X-ray is not recommended with this product.

CLEANING INSTRUCTIONS:

The Flo-Form Positioner is a cleanable and reusable product.

- Disinfect by wiping the casing with a hospital-approved disinfectant. Allow to dry.
- After disinfection, wipe down using a clean cloth moistened with clean water.
- Allow positioner to dry fully before use.
- Straight bleaching, scrubbing, machine washing, tumble drying, autoclaving not recommended.
- The disposable cover is single patient use and should be discarded appropriately. ②

For Flo-Form cloth covers (Available Accessory):

- Machine wash and dry at temperature up to 71°C.   
- Non-chlorine bleach is recommended.

INSTRUCTIONS FOR USE:

The Flo-Form Positioner effectively redistributes pressure over a greater surface area and promotes neutral spine and limb alignment. The positioner is moldable and can be adjusted to accommodate each patient's unique clinical needs.

- Take the positioner out of the plastic bag. Flo-Form positioners can be safely / evenly pre-warmed to the ambient incubator temperature by placing it in the end space of the baby's incubator (or on an unoccupied pre-warming admit bed) before positioning the baby.
- Place the pre-warmed fluidized positioner with cover under the patient.
- Gently stabilize the baby on the positioner with one hand, then slowly push the perimeter of the positioner towards the baby.
- Remold as needed for medical equipment and position changes.

WARRANTY:

Limited 90-day warranty against manufacturer defects.

STORAGE AND DISPOSAL:

No special requirements.

Rev 1:3

