


# Disposable Snuggly ROO™

*Not made with natural latex rubber. DEHP and BPA free.*

## INTENDED USE / CLINICAL BENEFIT:

The Disposable Snuggly ROO is a neurodevelopmentally supportive positioning aid that can be used for hospitalized preterm and ill infants in prone, supine, or side-lying positions.

## WARNINGS / CAUTIONS:

- **Hospital use only.**
- **Do not place straps around the baby's neck or face or use in any manner that might restrict infant's airway.**
- **Single patient use. Do not re-use between patients.** 

## CLEANING INSTRUCTIONS:

Discard appropriately after the product becomes soiled or after patient use. Do not re-use between patients.

## INSTRUCTIONS FOR USE:

- Select the correct size Disposable Snuggly ROO according to the infant's weight and body proportions. See label for size recommendations by weight.
- Put the Disposable Snuggly ROO in the incubator, warmer or crib with the footroll at the bottom of the bed and the straps off to the sides.
- Place the baby in the Disposable Snuggly ROO in a prone, supine, or side-lying position so the baby's legs are flexed appropriately and tucked inside the footroll, and the head and spine are in neutral alignment.
- Place the baby's hands near the face in a flexed position.
- Pull one strap over the baby's shoulder and arm to help hold the baby's hands in a comfortable, flexed position near his face.
- Tuck the strap under the Disposable Snuggly ROO so the weight of the baby will hold it in place.
- Pull the strap on the opposing side over the shoulder and arm to assist with maintaining appropriate flexion and tuck under. In side-lying, the second strap may not be needed.
- In order to maintain the desired position, the caregiver may use a Polly PAL™, Dandle® PAL, or Cozy Cub™ to give the baby support at the head and keep the baby from kicking out the footroll.
- Check all lines and tubes after positioning the baby to ensure proper and secure placement.

**NOTE:** If greater visualization of the baby is needed, the straps may be placed off to the side.

## STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments or to report any issue with the use of our product.

