

Recommended Laundering Procedures of Dandle•LION Products

Wash:

- Water temperatures up to 160 degrees Fahrenheit (71 degrees C).
- To avoid graying and fading, and degradation of high performance fabrics, only use peroxide (“color-safe”) bleach.
- Fasten Velcro® closures before washing.

Dry:

- Dry at temperatures up to 160 degrees Fahrenheit.

For optimal safety and performance, Dandle•LION recommends following the guidelines set by the CDC for laundering in a hospital setting. These guidelines indicate that hospitals should wash at 160 degrees Fahrenheit without chlorine bleach or at lower temperatures if chlorine bleach is added.

[CDC Guidelines](#)

(<https://www.cdc.gov/infection-control/media/pdfs/Guideline-Environmental-H.pdf>) | 4. Parameters of the Laundry Process; page 116)

IMPORTANT: The use of chlorine bleach will reduce product life by causing fading and degradation of the high performance fabrics and Velcro over time, therefore we strongly recommend a non-chlorine bleach if it is needed.