

Dandle[®]PURE Neonatal Mattress

Not made with natural latex rubber. DEHP free.

INTENDED USE:

A neonatal mattress designed to support the body during sleep or rest.

WARNINGS / CAUTIONS:

- Hospital use only.
- To prevent cross-contamination between patients, clean according to hospital policies.
- Cannot be repaired. Inspect the mattress for damage (punctured, torn, seals opened, etc.) and discard if necessary.
- To prevent suffocation or entanglement, never use crib sheet unless it fits securely on mattress, and ensure baby's face is not immersed into the mattress.

CLEANING INSTRUCTIONS:

The DandlePURE Mattress is suitable for multi-patient use.

- Clean the mattress after each patient discharge or when soiled.
- Ensure that all cleaning and disinfection agents used comply with the Approved Disinfectants list on our website.
- Wait for mattress to dry.

Note:

- Avoid abrasive scrubbing of the filter and fabric.
- Do not bleach, machine wash, dry clean, tumble dry, or iron the mattress.
- The use of alcohol wipes and cleaning solutions can temporarily change the surface of the cover fabric and make it more susceptible to mechanical damage.



INSTRUCTIONS FOR USE:

- Inspect mattress before use to ensure that:
 - No holes or damage are apparent.
 - The PneumaPure[®] Filter is firmly attached to the mattress casing and not compromised.
 - All mattress casing seams are closed, and no tears or holes are present.
 - The mattress is free from stains and unpleasant odor.
 - To prevent infant from bottoming out, ensure the base of the bed cannot be felt through the foam.

The mattress should be discarded and replaced if it does not meet the criteria above.

- A securely-fitting, breathable sheet is recommended over the DandlePURE Mattress.
- The use of developmental care products, such as Dandle•LION positioning aids, are recommended with any DandlePURE Mattress.

WARRANTY:

Limited 2-year warranty against manufacturer defects.

STORAGE AND DISPOSAL:

- Store mattress flat or on its side.

Please contact Dandle•LION Medical with any questions or comments
or to report any issue with the use of our product.

Rev 1:0924