

# Dandle<sup>®</sup>PURE Neonatal Mattress

Not made with natural latex rubber. DEHP free.

## INTENDED USE:

A neonatal mattress designed to support the body during sleep or rest.

## WARNINGS / CAUTIONS:

- Hospital use only.
- To prevent cross-contamination between patients, clean according to hospital policies.
- Cannot be repaired. Inspect the mattress for damage (punctured, torn, seals opened, etc.) and discard if necessary.
- To prevent suffocation and/or entanglement, use securely fitted linen to cover the mattress and ensure the patient is not lying face-down.


## CLEANING INSTRUCTIONS:

The DandlePURE Mattress is suitable for multi-patient use.

- Clean the mattress between patients and/or when soiled.
- Approved cleaning solutions are listed in the [DandlePURE Neonatal Mattress Approved Disinfectants](#) resource.
- Allow mattress to dry completely before use.

## Note:

- Avoid abrasive scrubbing of the filter and fabric.
- Do not machine wash, dry clean, tumble dry, or iron the mattress.
- Use of undiluted bleach is not recommended.
- Chlorine can damage the surface fabric of the product during long-term use. After using disinfectants containing chlorine, it is recommended to wipe the surfaces with a damp cloth.

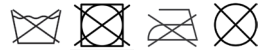

## INSTRUCTIONS FOR USE:

- Inspect mattress before use to ensure that:
  - No holes or damage are apparent.
  - The PneumaPure<sup>®</sup> Filter is firmly attached to the mattress casing and not compromised.
  - All mattress casing seams are closed, and no tears or holes are present.
  - The mattress is free from stains and unpleasant odor.
  - Mattress surface is flat and free from indentations.
- The mattress should be discarded and replaced if it does not meet the criteria above.
- To prevent infant from bottoming out, ensure the base of the bed cannot be felt through the foam.
- Mattress should be covered with a single layer of securely fitting linen.
- Developmental care products, such as Dandle•LION positioning aids, may be used with any DandlePURE Mattress as appropriate to patient need.

## WARRANTY:

Limited 2-year warranty against manufacturer defects.

## STORAGE AND DISPOSAL:

Store mattress flat or on its side.

No special disposal requirements.

Please contact Dandle•LION Medical with any questions or comments  
or to report any issue with the use of our product.

Rev 2:1124