

Mini PAL™ - Wipeable

Not made with natural latex rubber. DEHP free.




INTENDED USE:

Provide positioning support for infants, especially ELBW (Extremely Low Birth Weight) infants, in prone, supine, and side-lying positions.

WARNINGS / CAUTIONS:

- Hospital use only.
- Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.
- Special caution should be taken to avoid placing the full weight or too many micro discs on top of small infants.
- To prevent cross-contamination between patients, clean according to hospital policies.
- To prevent micro discs from escaping the product and posing a hazard to the baby, and to prevent ingress of contaminants into the product, regularly inspect the wipeable fabric for any visible signs of damage or wear. Discard immediately if the exterior of the product is damaged or compromised in any way.
- Always use with cover.

CLEANING INSTRUCTIONS:

- The Mini PAL is a wipeable and reusable product that should be wiped clean with a standard hospital quaternary disinfectant any time it becomes soiled and between patient use.
- The disposable cover is single use and should be discarded immediately. 

For Mini PAL cloth covers (Available Accessory):

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



INSTRUCTIONS FOR USE:

The Mini PAL is a flexible positioning aid that can be used in a variety of ways for all babies and was developed intentionally for babies < 800 g. When used inside the XS Dandle® ROO2, Dandle® ROO Lite, or Dandle® WRAP Stretch, the Mini PAL will fill the space at the bottom of the pouch area to improve product functionality.

- In supine, with the baby's head up against the headroll, place the larger base piece ("body") in the space from the baby's bottom to where the pouch meets the base. The baby's legs should rest on top of the "body" in a posterior pelvic tilt. Use the two symmetric pieces that extend from the "body" ("legs") to snuggle up beside the torso, under the arms, filling up space and helping to keep hips and shoulders in flexion. With the Mini PAL inside the positioner, the wings and pouch will align properly.
- With the baby in a side-lying position, place the "body" behind the baby and put the lightweight "legs" over the baby's shoulders and hips to maintain position or provide comfort.
- May be used with larger babies to help support proper positioning, provide proprioceptive feedback, or support tubing/equipment.
- If no headroll is available and midline head positioning is warranted, place Mini PAL around infant's head with "legs" beneath baby's shoulders, supporting arms in flexion.
- If limb extension is necessary, place the "body" under the joint and the "legs" above and below the joint to hold it in extension.

STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments or to report any issue with the use of our product.

Rev 2:0924



Tarry Medical Products Inc.
dba Dandle•LION Medical
www.dandleLIONmedical.com

22 Shelter Rock Lane
203-791-9001

Danbury, CT 06810
© DandleLION Medical 2021