

Cozy Cub™

Not made with natural latex rubber. DEHP free.



INTENDED USE / CLINICAL BENEFIT:

The Cozy Cub is a reusable positioning boundary, in three sizes, that can be used for hospitalized infants.

WARNINGS / CAUTIONS:

- Do not place near infant's face or use in any manner that might restrict the airway.
- Always use with a cover.
- Keep 'loop' at end of cover away from infant's fingers or toes.
- Hospital use only.
- DO NOT stick needles into the Cozy Cub through the cover.
- Special caution should be taken to avoid placing any part of the Cozy Cub on top of any part of the infant.
- Regularly inspect the Cozy Cub for any visible signs of damage or wear such as cuts, tears, cracks, pinholes, snags, or stains.
- Immediately discard if Cozy Cub has any visible signs of stains, damage, or wear.

CLEANING INSTRUCTIONS:

- The Cozy Cub is wipeable with a standard hospital quaternary disinfectant and reusable.
- The disposable cover is single patient use and should be discarded appropriately.



For Cozy Cub cloth covers (Available Accessory):

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



INSTRUCTIONS FOR USE:

The Cozy Cub is a flexible positioning aid that can be used for infants with a variety of medical conditions. Regardless of position, follow the 5 Tenets™ for proper positioning: flexion, containment, midline alignment, 360° of proprioception, and free movement and recoil.

The Cozy Cub provides containment while allowing for full visualization of the infant. It can be used as a complete boundary or folded away from the baby for partial support. The 'loop' at the end of the cover can be used to make a more secure, smaller boundary by pulling the opposing end through the loop. The internal silky discs can be moved around to provide more or less support for infant or any medical equipment.

- Supine: Provide full or partial boundary around the infant. Be sure to support shoulders, hips, and legs to encourage proper flexion.
- Side-lying: Utilize Cozy Cub to prevent infant from rolling over, to keep hands to face, and to provide flexion in the lower extremities.
- Prone: Ensure head and neck are in alignment, trunk is supported, and hips are drawn into posterior pelvic tilt. Use the Cozy Cub to maintain infant in this flexed and contained position.
- Ideal for babies whose condition requires head of the bed elevation.

STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments
or to report any issue with the use of our product.



Tarry Medical Products Inc.
dba DandleLION Medical
22 Shelter Rock Lane
Danbury, CT 06810 USA
+1 203-791-9001

www.dandleLIONmedical.com
© DandleLION Medical 2009

EC REP

Advena Medical
Tower Business Centre
2nd Floor, Tower Street
Swatar BKR 4013 Malta

Rev 4:0225