

# Snuggly ROO™

*Not made with natural latex rubber. DEHP and BPA free.*



## INTENDED USE / CLINICAL BENEFIT:

The Snuggly ROO is a neurodevelopmentally supportive positioning aid that can be used for hospitalized preterm and ill infants in prone, supine, or side-lying positions.

## WARNINGS / CAUTIONS:

- **Hospital use only.**
- **Do not place straps around the baby's neck or face or use in any manner that might restrict infant's airway.**

## CLEANING INSTRUCTIONS:

Machine wash and dry all cloth components at temperatures up to 71°C. Higher temperatures will cause additional shrinkage. Visually inspect product after each laundering.



## INSTRUCTIONS FOR USE:

- Select the correct size Snuggly ROO according to the infant's weight and body proportions. See label for size recommendations by weight.
- Put the Snuggly ROO in the incubator, warmer or crib with the footroll at the bottom of the bed and the straps off to the sides.
- Place the baby in the Snuggly ROO in a prone, supine, or side-lying position so the baby's legs are flexed appropriately and tucked inside the footroll, and the head and spine are in neutral alignment.
- Place the baby's hands near the face in a flexed position.
- Pull one strap over the baby's shoulder and arm to help hold the baby's hands in a comfortable, flexed position near his face.
- Tuck the strap under the Snuggly ROO so the weight of the baby will hold it in place.
- Pull the strap on the opposing side over the shoulder and arm to assist with maintaining appropriate flexion and tuck under. In side-lying, the second strap may not be needed.
- In order to maintain the desired position, the caregiver may use a Polly PAL™, Dandle® PAL, or Cozy Cub™ to give the baby support at the head and keep the baby from kicking out the footroll.
- Check all lines and tubes after positioning the baby to ensure proper and secure placement.

**NOTE:** If greater visualization of the baby is needed, the straps may be placed off to the side.

## STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments  
or to report any issue with the use of our product.

Rev 2:0325



Tarry Medical Products Inc.  
dba DandleLION Medical  
[www.dandleLIONmedical.com](http://www.dandleLIONmedical.com)

22 Shelter Rock Lane  
203-791-9001

Danbury, CT 06810  
© DandleLION Medical 2021