

Dandle® PAL - Wipeable



Not made with natural latex rubber. DEHP free.

More information is available online at www.dandleLIONmedical.com


INTENDED USE:

Provides containment and supports proper positioning of hospitalized infants in prone, supine, and side-lying positions. May also be used to support the positioning of medical equipment inside the infant's bed.

WARNINGS / CAUTIONS:

- Hospital use only.
- Special caution should be taken to avoid placing the full weight or too many micro discs on top of small infants.
- Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.
- To prevent cross-contamination between patients, clean according to hospital policies.
- To prevent growth of any internal contaminant, regularly inspect the wipeable covering for any visible signs of damage or wear. Discard if necessary.
- If the outside of the product becomes punctured or otherwise compromised, discard immediately.

CLEANING INSTRUCTIONS:

- The Dandle PAL is a wipeable and reusable product that should be wiped clean with a standard hospital quaternary disinfectant any time it becomes soiled and between patient use.
- The disposable cover is single patient use and should be discarded immediately. 

For Dandle PAL cloth covers (Available Accessory):

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



INSTRUCTIONS FOR USE:

The Dandle PAL is a flexible positioning aid that can be used in a variety of ways. The PAL has two internal "beanbags": a large bag that fills the horseshoe and a smaller bag at the rounded top.

- The horseshoe part, or "legs," can be placed around the baby's head or feet to create a boundary, help maintain a position, or give a feeling of security.
- The smaller bag can be placed on top of the larger one if a higher boundary is needed. The smaller bag can be held in place by pulling the strap under the horseshoe part.
- The horseshoe can be opened up wide and put behind a baby in side-lying position to help maintain the position and keep the baby in flexion. The smaller bag may be strapped on top of the larger one to help keep the baby from rolling back.
- The smaller bag can be used to support tubing.
- The micro discs can be shifted inside the "legs" to make them fuller or emptier to best support the baby's medical and developmental needs.

STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments
or to report any issue with the use of our product.

Rev 4:0525



Tarry Medical Products Inc.
dba Dandle•LION Medical
www.dandleLIONmedical.com

22 Shelter Rock Lane
203-791-9001

Danbury, CT 06810
© DandleLION Medical 2021