

Polly PAL™



Not made with natural latex rubber. DEHP free.

More information is available online at www.dandleLIONmedical.com

INTENDED USE:

Aids clinicians in properly positioning hospitalized infants in prone, supine, and side-lying positions. May also be used to support the positioning of medical equipment inside the infant's bed.

WARNINGS / CAUTIONS:

- Hospital use only.
- Special caution should be taken to avoid placing the full weight or too many micro discs on top of small infants.
- Do not place near the baby's neck or face or use in any manner that might restrict baby's airway.
- To prevent cross-contamination between patients, clean according to hospital policies.
- If micro discs should ever become loose or if a hole occurs in the fabric cover, discard immediately.

CLEANING INSTRUCTIONS:

The Polly PAL is a washable and reusable product.

- Machine wash and dry at temperature up to 71°C.
- Non-chlorine bleach is recommended.



INSTRUCTIONS FOR USE:

The Polly PAL is a flexible positioning aid that can be used in a variety of ways.

- Place around the baby's head or feet to create a boundary, help to maintain a position, or give a feeling of security.
- Use it to prop a pacifier or support tubing.
- Move the micro discs in different places to make parts or sections fuller or less full to accommodate equipment or the baby's positioning needs.

STORAGE AND DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments
or to report any issue with the use of our product.



Rev 2:0525



Tarry Medical Products Inc.
dba Dandle•LION Medical
22 Shelter Rock Lane
Danbury, CT 06810 USA
+1 203-791-9001

www.dandleLIONmedical.com
© DandleLION Medical 2009



Advena Medical
Tower Business Centre
2nd Floor, Tower Street
Swatara BKR 4013 Malta